

MESO PEEL

Step by Step

Meso Peel

EN-2020



mesoskinline[®]

MILANO - HAMBURG - COPENHAGEN *By Aleksandra Hjaersfeldt*

MESO PEEL

ADVANCED PEELING RANGE



MESO PEEL MELANIN

Anti-photoaging and hyperpigmentation properties

Pyruvic acid 20% + Resorcinol 10% + Phytic acid 6%

Pyruvic Acid is effective in the reduction of melasma, post-inflammatory hyperpigmentation, plays an important depigmenting and inhibitory action against keratinocytes. It has antibacterial, anti-inflammatory and seboregulatory properties.

Resorcinol is a phenolic derivative with keratolytic and regenerating properties, it fights melanin hyperpigmentations and has a whitening effect.

Phytic Acid powerful chelator of copper that encourages tyrosinase enzyme inhibition and has a depigmenting effect.



Each pack contains a neutralizing post-peel solution.

MESO PEEL MELANIN

- 1 x 50 ml bottle Meso Peel Melanin
 - pH 3.0
 - Hydroalcoholic gel
- 1 x 10 ml jar of Protector
- 1 x 25 ml bottle of PeelStop

INDICATIONS

- superficial and epidermal pigmented lesions
- solar lentigos
- ephelides
- dull skin or skin with uneven tone
- it is suitable for areas of the body with hyperpigmentation

it is necessary to prescribe a home treatment to optimise the result (Meso Lift and Protect and Meso Pigment Reduce)

MESO PEEL ACNE

Anti-fungal and anti-acne properties

Pyruvic acid 20% + Salicylic acid 20%

Pyruvic Acid is effective in the reduction of melasma, post-inflammatory hyperpigmentation, plays an important depigmenting and inhibitory action against keratinocytes. It has antibacterial, anti-inflammatory and seboregulatory properties.

Salicylic Acid is a low hydrophilic beta-hydroxyacid, acting as a keratolytic and sebum regulator.

It has anti-inflammatory, antimicrobial, astringent and anti-ageing properties.



Each pack contains a neutralizing post-peel solution.

MESO PEEL ACNE

- 1 x 50 ml bottle Meso Peel Acne
 - pH 3.0
 - Hydroalcoholic gel
- 1 x 10 ml jar of Protector
- 1 x 25 ml bottle of PeelStop

INDICATIONS

- actinic keratosis
- comedonic and inflammatory acne
- middle-grade acne scars
- hyperpigmentation
- rosacea

it is necessary to prescribe a home treatment to optimise the result (Meso Lift and Protect and Meso Pigment Reduce)

IMPORTANT

ALWAYS store the Peeling Kit in a refrigerator and in the original alu bag to avoid daylight, this will maintain the PH value.

TIPS*

Recommended maximum application times (minutes):

The application in layers and the exposure time depend on the type of thickness, skin conditions and the desired therapeutic result.

| Cocktails | face | neck | body |
|-------------------|---------|--------|------------|
| Meso Peel Melanin | 5 min. | 2 min. | 5 - 7 min. |
| Meso Peel Acne | 10 min. | 2 min. | 5 min. |

CERTIFIED
mesoskinline®
ACADEMY



ACCREDITED
EDUCATION
PROGRAMMES

PROFESSIONAL USE: INSTRUCTIONS STEP BY STEP

ADVANCED PEELING RANGE

STEP 1: MESO PURE PEEL

Prepare the skin properly with MESO PURE PEEL before undergoing MESO PEEL treatment.

- Cleanse the area to be treated, carefully removing any makeup
- Pre-procedure cleansing solution / wipe (Meso Pure Peel)

Active ingredients

Lactic Acid, Citric Acid, Malic Acid, Bisabolol, Citrus Grandis Fruit Extract, Vaccinum Myrtillus Leaf Extract, Citrus Lemon Peel Extract



STEP 2: MESO PEEL PROTECTION

Protect sensitive areas (periocular area, lips, nasal mucosa)
Formula is based on moisturizing and protective agents.



STEP 3: APPLYING MESO PEEL

Apply a thin layer of MESO PEEL ACNE or MESO PEEL MELANIN to the whole area with a soft fan shaped brush and leave it on the skin according to application instructions for each area*.

| Cocktails | face | neck | body |
|-------------------|---------|--------|------------|
| Meso Peel Melanin | 5 min. | 2 min. | 5 - 7 min. |
| Meso Peel Acne | 10 min. | 2 min. | 5 min. |



STEP 4: MESO PEEL STOP

Remove the peel by neutralizing it with water and the Peel Stop on a cotton pad.
(mono ingredient or cocktail)



STEP 5: MESO ANTI-AGE MASK

Post-peel hydrogel patch mask (MESO ANTI-AGE MASK).

Leave the mask on for 15 min, for young skin and up to 40 min. for mature skin.

Hydrogel Face Mask, easy and fast to use (pre-cut). It's 3D reticular structures ensures maximum adhesion to the skin. It is dampened with active ingredients specially selected for treatment of different types of skin. Soothing, moisturizing the skin and does not cause irritation.



STEP 6: MESO LIFT AND PROTECT (SPF50+)

Moisturizing sun protection SPF50+ (MESO LIFT AND PROTECT).
Apply a thin layer on the treated area. Follow up with a light mineral powder if necessary.

MESO LIFT AND PROTECT is a photoprotector, suitable for everyday use. Ideal day cream for dry hyper-reactive skin, photodermatosis, photosensitivity.

MESO LIFT AND PROTECT is a multifunctional anti-age product with a strong action lasting up to 8 hours.



Headquarter, sales, distribution and showroom

mesoskinline®
mesoskinline ApS

Juelsmindevej 57
DK-7120 Vejle
Denmark

(+45) 71 74 11 08
sales@mesoskinline.com
www.mesoskinline.dk

Innovative
non-invasive
mesotherapy solutions



mesoskinline®

MILANO - HAMBURG - COPENHAGEN *By Aleksandra Kjaersfeldt*